# GIBAUD

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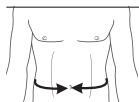
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

## INDICATIONS

For ribs conditions that may benefit from support, such as :

- Rib fracture.
- Costal trauma.

## PERFORMANCE

The device provides compression and rib cage support.  
The device provides pain relief.

## CONTRAINDICATION

Punctured lung.  
Unstable, displaced fractures.

## WARNINGS • PRECAUTIONS OF USE

This product contains Naturel Rubber Latex which may cause allergic reactions.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE

Belt composed of 3 elastic bands with hook-and-loop closure system.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

The fastening closure is to be positioned contralaterally to the injury. Adjust the closure of the belt to obtain maximum pain relief by firm but comfortable tightening.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

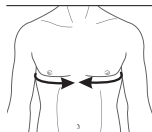
Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the chest circumference (in cm)

6098 : chest	70-100	100-130
Size (Men)	1	2
6085 : chest	70-95	95-120
Size (Women)	1	2

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

## INDICATIONS

For abdominal conditions that may benefit from compression, such as:

- Abdominal strain or weakness of the abdominal area.

## PERFORMANCE

The devices provide abdominal compression and/or support to the abdomen.

The devices provide pain relief.

The devices provide respiratory function improvement in patients with spinal cord injury.

The devices provide physical function improvement (walking distance) in patients with major abdominal surgery.

## CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

## WARNINGS • PRECAUTIONS OF USE

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

This product contains Naturel Rubber Latex which may cause allergic reactions.

Do not wear for extended periods without medical follow-up

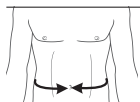
The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

## MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the waist circumference, measured 2cm above the iliac crests (cm)

Size	1	2	3	4	5
6072	70-80	80-90	90-100	110-110	-
6193	61-75	76-90	91-105	106-120	121-135

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE

The elastic support belt is a band stretchable in the lengthwise direction. The belt has a standard height of 27 cm in the back and 21.5 cm in the front.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

1- Position the belt around the abdomen so that the hook-and-loop fastening system is at the front.

2- Adjust the length of the belt to obtain a firm but comfortable tightening.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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GIBAUD S.A.S.U